

SALTER LABS®



NEW

Salter High Flow Humidifier (6-15 LPM)

Model #7900

350 cc Bubble Humidifier - Dry

A new pressure relief valve and down spout diffuser design allow high flow delivery (6-15 LPM). This humidifier is specifically suited for applications where delivery via face mask or the new Salter-Style® 1600HF cannula require high oxygen flow rates to maintain desired oxygen saturation.

Ideal for use with new generation high capacity output oxygen concentrators, wall outlet source gas or other applications where flow rates up to 15 LPM are required.



Features

- NEW 360° molded low resistance high output diffuser head design
- NEW special diffuser and relief valve design, delivers high flows with minimum internal resistance
- NEW pressure relief valve that automatically resets
- Audible, pressure relief alert
- Large “quad style” wingnut connector
- Jar permanently marked in large print, minimum maximum water levels
- Domed lid is two tone beige and black

Benefits

- Quiet, efficient hydration operation; functional range 6-15 LPM
- Less water turbulence and delivery tube rain out. Eliminates water spillage through relief valve
- Reduces nurse / caregiver time in case of temporary occlusion
- Warns of potential patient or equipment hazards
- Easy to grip, reduces chance of cross threading
- Easy to read, durable, will not rub, wear or wash off
- Distinctive color to signify this as a high flow humidifier, model #7900

Variety of uses... Homecare, Transport, Hospital, Extended Care Facilities, Hospice



High Flow

Salter Bubble Humidifier

Model #7900 (6-15 LPM)


A new generation humidifier for applications requiring flow rates of 6 to 15 LPM. The quiet operation is ideal in any setting (Hospital, Hospice, or Homecare). All new design relief valve will audibly alert of potential downstream occlusions. It automatically resets and continues gas flow. A unique new 360° rectangular hole diffuser provides superior gas hydration and quiet operation with very low resistance, full range 6-15 LPM.

This model *high flow* #7900 humidifier is uniquely suited to enhance gas hydration for the new generation oxygen concentrators. It can be utilized effectively with patients requiring high flows delivered via masks and the new Salter-Style® 1600HF nasal cannula. The domed lid is beige and black to readily denote that a high flow system is in operation.

Disposable Humidifiers from Salter Labs are “truly single patient use products designed and engineered for long term durability and complete patient satisfaction”.

Large, easy to grip quad style wing nut. Resists cross threading, simplifies secure connection to oxygen supply source.

Audible safety pressure pop-off alarm. Automatically resets

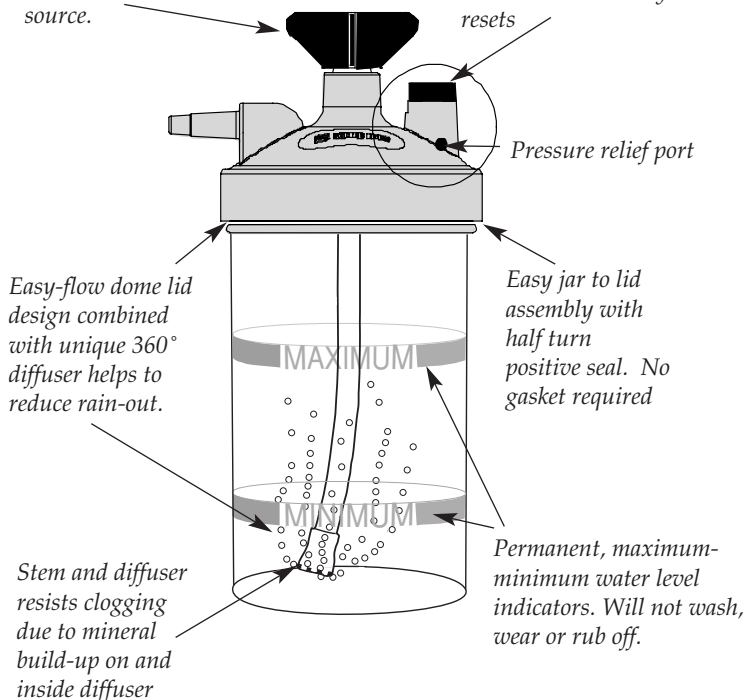


Easy-flow dome lid design combined with unique 360° diffuser helps to reduce rain-out.

Easy jar to lid assembly with half turn positive seal. No gasket required

Stem and diffuser resists clogging due to mineral build-up on and inside diffuser


Permanent, maximum-minimum water level indicators. Will not wash, wear or rub off.

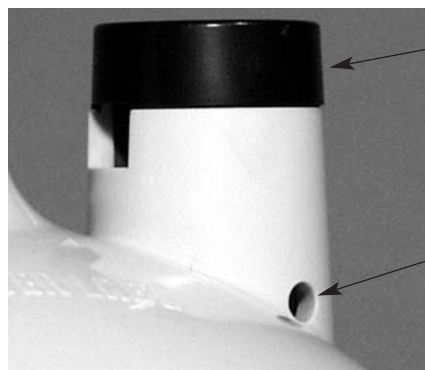

Relief Poppet Housing Assembly

Pressure Relief Port

Two special pressure ports allow poppet weight to automatically reset. It will recycle until obstruction downstream is cleared.

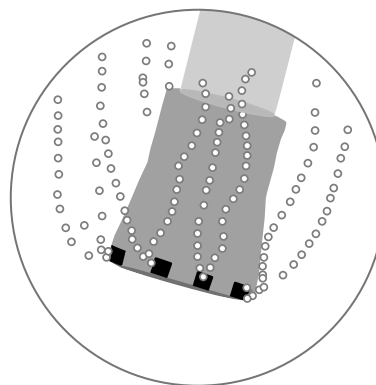

Unique 360° rectangular diffusion ports operate quietly with improved hydration and less system back pressure.

Ordering Information

	Units (per case)	Part No.
<i>High Flow Model #7900 Bubble Humidifier-Dry</i>		
<i>High Flow 350 cc Bubble Humidifier (for flows from 6-15 LPM)</i>	10 or 25	7900

SALTER LABS®

100 W. Sycamore Road - Arvin, CA 93203
 Phone: 661-854-3166
 1-800-421-0024 (CA only)
 1-800-235-4203 (U.S. & Canada)
 Fax: 661-854-3850
 Toll Free Fax: 1-800-628-4690 (U.S. & Canada)
 International Fax: 661-854-3850
 E-mail: salterlabs@us.salterlabs.com
 www.salterlabs.com

CE
0482

ISO 9001/EN 46001 & ISO 13485